

## OBTAINING ANIMAL HAIR FOR ANALYSIS

Obtaining a hair sample for analysis is the same for animals as it is for humans. We need 125 mgs (about ½ tablespoon) taken as close to the skin as possible. The hair should be no longer than 1 ½ inches. For long haired cats and dogs, the extra length should be cut off and discarded.

The hair should be taken from the animal's chest area. For animals who are mainly outdoors, wash the testing area with rubbing alcohol and let dry completely before taking the sample.

The sample can be contained in a paper envelope, such as provided by the lab, or a regular mailing envelope. Plastic baggies are not suggested inasmuch as it is difficult to retrieve all of the hair sample. **Never** tape the hair sample to a piece of paper or store it in aluminum foil inasmuch as this will contaminate the sample.

Animal analysis results consist of readings only. We do not provide supplement recommendations or an interpretation.

Hair testing kits for animals are provided from Analytical Research Labs, Inc. Please call 1-800-848-2667 to request that supplies be sent or for any further information.

**ANIMAL HAIR ANALYSIS INFORMATION FORM**

Client Account # \_\_\_\_\_

Date \_\_\_\_\_

Lab # \_\_\_\_\_

Name of Animal \_\_\_\_\_

Type of Animal \_\_\_\_\_

Age \_\_\_\_\_ Weight \_\_\_\_\_ Sex \_\_\_\_\_ Pregnant? Y \_\_\_\_\_ N \_\_\_\_\_

Date of Hair Sampling \_\_\_\_\_

Color of Hair \_\_\_\_\_

Sample Taken from what Location on Body? \_\_\_\_\_

Owner's Name \_\_\_\_\_

Address \_\_\_\_\_

City \_\_\_\_\_ State \_\_\_\_\_ Zip \_\_\_\_\_

Telephone # \_\_\_\_\_

Client's Name \_\_\_\_\_

Office Address \_\_\_\_\_

City \_\_\_\_\_ State \_\_\_\_\_ Zip \_\_\_\_\_

Telephone # \_\_\_\_\_

**PROFILE 1 ONLY**

