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Firstbeat Life shows you the state of your well-being and helps you achieve things that are important to you.

Firstbeat Life produces uniquely accurate information about your well-being. You will see how your body reacts to different situations and learn to understand **why**. You will identify behaviours that support or hinder your well-being and can learn the **causes** behind overload or poor recovery. The service helps you achieve things that you value.

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17:24 🖪 🔌 📚 .il 45%	17:25 🖻 🛛 🔌 南네 45% 🛢	17:25 🖪 🛛 🔌 응: 대 44%
< Sign up	< Sign up	< Sign up
Terms of service and privacy policy Firstbeat Technologies Oy takes our privacy and personal data seriously. By signing up to Firstbeat Life, you agree to our Service End User Terms, End User License Agreement and Privacy Policy.	Add your email address XXX.XXX @gmail.com Use your personal and permanent email address to ensure that you can access your data at all times.	Set account password Markov Content of the set of the
✓ I understand and agree the Service End User Terms and EULA (links below).	Set account password	Your date of birth
End User License Agreement	<u></u>	15/04/1992
Service End User Terms When using the measurement device and taking part in the Firstbeat Life Service, I understand and I agree that it is mandatory	Password must have at least 10 characters, contain uppercase & lowercase letters and a number to help ensuring your data privacy.	According to the Terms of Service, any person under the age of 16 is prohibited from signing up and using this service.
to provide personal data and health related data (including heart rate data, illnesses and medication) in order to conduct the assessment. By ticking this box I understand and I agree that the formation that I	Your date of birth 01/01/1985 →	$\rightarrow$
	III O <	
17:25 🕅 🖻 🛛 🔌 🖘 💵 44%	17:27 🖪 🔌 📚 ال 44% 🚔	17:28 🖻 🔌 즉, dl 44% 🛢
< Sign up	< Sign up	← Sign up
Verify your email address	Verify your email address	
Enter verification code	Enter verification code	Checking license information
Verification code was sent to your email address xxx.xxx @gmail.com	Verification code was sent to your email address <b>xxx.xxx</b> @gmail.com	
Didn't get the code? <u>Resend code</u>	Didn't get the code? <u>Resend code</u>	
Marife an -il	Vorificament	
verny email	Verny email	

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#### < Activate license

#### **Firstbeat Life**

# Consent to the sharing of your personal data

You are activating your Firstbeat Life subscription offered to you by **Biocoherence Nederland**. To provide their service to you in the best possible way, the Biocoherence Nederland needs certain personal data you have stored in Firstbeat Life, such as new and previously obtained Firstbeat Life measurement results (your heart rate stress, recovery, sleep, exercise and aerobic fitness results).

By activating the subscription, I consent for Firstbeat Technologies Oy to share my personal data and measurement results stored in Firstbeat Life to Biocoherence Nederland.

```
List of personal data to be transferred
```

The Biocoherence Nederland can save a copy of the personal data they receive in their own systems. Contact Biocoherence Nederland for more Agree

Your Firstbeat Life account has been created. Start using the service by following the instructions below.

17:29 🖻 🔌 🧙 🚓 州 44% 🛢	17:29 🖪 🔌 🧙 🖘 🖿 44% 🖿
× Settings	Firstbeat Life
Account security & settings <b>Email Change XXX.XXX Qgmail.com</b>	
First name xxxxxxx	Your Firstbeat Life account has been created. Start using the service by following the instructions below.
Last name xxxxxxx	Share error messages and usage dataTo keep improving the quality of the service, we ask for your permission to collect anonymous usage data.AllowDo not allow
Phone number	Initial steps
	Image: home     Image: home     Image: home     Image: home       Home     Results     Insights     Profile       Image: home     Image: home     Image: home     Image: home

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17:32	2 🖻 🔌 🖏 🔐 42% 🗖	17:33 🖪	🗙 🗟 וו 42%
<	Firmware update	Updating firmware	succeeded
	- S C Real		
kee you ei wł	A new firmware update is available. By eping your measurement device updated, u will have access to new features and can nsure reliable functioning of the device. Remember to update the firmware nenever a new version is available to take full advantage of all the features.	You have the la your me Cu La	test firmware installed on asurement device. rrent version 84 utest version
	Current version <b>79.36</b>		04
	Latest version <b>84.39</b>		
	Update firmware		
	III O <	Ш	0 <

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